**TRACOE *vario* Tracheostomy Tubes
with the Minimally Traumatic Insertion System**

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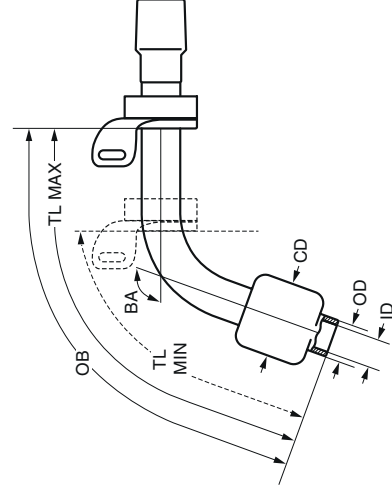
Size Table

REF 450-P		REF 460-P		REF 470-P	
Size	TL (min-max) mm	TL (min-max) mm	TL (min-max) mm	Size	TL (min-max) mm
07	53.0 – 68.0	54.0 – 69.0	54.0 – 69.0	07	63.0 – 83.0
08	64.0 – 84.0	65.0 – 85.0	65.0 – 85.0	08	64.0 – 91.0
09	66.0 – 93.0	64.0 – 91.0	64.0 – 91.0	09	64.0 – 91.0

REF 451-P		REF 461-P		REF 471-P	
Size	TL (min-max) mm	TL (min-max) mm	TL (min-max) mm	Size	TL (min-max) mm
07	65.0 – 98.0	64.5 – 97.5	64.5 – 97.5	07	73.0 – 113.0
08	76.0 – 116.0	73.5 – 113.5	73.5 – 113.5	08	73.5 – 118.5
09	73.5 – 118.5	78.0 – 123.0	78.0 – 123.0	09	73.5 – 118.5

ID: inside diameter (clear width) at bottom of cannula; **OD:** outside diameter at bottom of cannula; **TLmin:** minimal length along centre line; **TLmax:** maximal length along centre line; **OB:** length along outer bend; **BA:** bending angle; **CD:** cut diameter

DE / Achtung. Gebrauchsanweisung beachten · **EN** / Warning. Please read Instructions for Use · **FR** / Attention. Respecter le mode d'emploi · **IT** / Attenzione. Consultare le istruzioni per l'uso · **ES** / Atención. Siga las instrucciones de uso · **PT** / Atenção. Ter atenção às instruções de utilização · **DA** / Vigtigt: Følg brugsanvisningen · **FI** / Huomio. Noudata käyttöohjetta · **NO** / OBS. Følg bruksanvisningen · **HU** / Figyelem. Olvassa el a használati útmutatót · **SV** / OBS! Se bruksanvisningen · **NL** / Let op. Rekening houden met de gebruiksaanwijzing · **HR** / Upozorenje. Pridržavajte se uputa za uporabu · **SL** / Pozor. Upoštevajte navodila za uporabo · **CS** / Pozor. Respektujte návod k použití · **RO** / Atenție. Respectați instrucțiunile de utilizare · **TR** / Dikkat. Kullanma talimatına dikkat ediniz · **RU** / Внимание! Соблюдать инструкцию по применению · **PL** / Ostrzeżenie. Zastrzyż do instrukcji używania · **EL** / Προσοχή. Ακολουθήστε τις οδηγίες χρήσης · **KO** / 주의. 사용 설명서를 준수하십시오 · **ZH** / 注意. 注意使用说明书
אזהרה. עקוב אחר הוראות השימוש / **AR** / تنبيه. يجب مراعاة دليل الاستعمال / **HE**

DE / Medizinprodukt · **EN** / Medical Device · **FR** / Dispositif médical · **IT** / Dispositivo medico · **ES** / Dispositivo médico · **PT** / Dispositivo médico · **DA** / Medicinsk udstyr · **FI** / Lääkinnällinen laite · **NO** / Medisinsk enhet · **HU** / Orvosi eszköz · **SV** / Medicinteknisk produkt · **NL** / Medisch hulpmiddel · **HR** / Medicinski proizvod · **SL** / Medicinski pripomoček · **CS** / Zdravotnický prostředek · **RO** / Dispozitiv medical · **TR** / Tıbbi cihaz · **RU** / Медицинское изделие · **PL** / Wyrób medyczny · **EL** / Ιατροτεχνολογικό προϊόν · **KO** / 의료 기기 · **ZH** / 医疗器械 · **HE** / מכשור רפואי / **AR** / جهاز طبي

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Instructions for Use

TRACOE *vario* Tracheostomy Tubes with the Minimally Traumatic Insertion System

Note: Please read the instructions for use carefully. They are part of the described product and must be available at all times. For your patients' and your own safety, please observe the following safety information.

Fold out pages: The illustrations to which the text refers can be found on the (fold-out) illustrated pages at the beginning of these instructions. The numbers indicate product components and refer to the respective illustrations of the product. Symbols and icons used with the product are explained in sections "General Description" and "Functional Description".

1. Intended Use and Indications for Use

TRACOE *vario* tracheostomy tubes are indicated for providing tracheal access for airway management especially to those patients with unusual anatomy or patients with thick necks. They may be used for up to 29 days.

Clinical Benefit: TRACOE *vario* tracheostomy tubes provide tracheal access to the lower respiratory tract. The cuffed models, when inflated, can be used to seal the airway (e.g. for mechanical ventilation).

The tubes with subglottic suction line allow to remove the secretions that remain above the inflated cuff.

The adjustable neck flange provides the option to vary the proximal length of the tube (e.g. for patients with extended stoma channel).

The tracheostomy tubes with additional distal length (*vario* XL) can be used to position the low-pressure cuff further in the caudal direction (e.g. to stent tracheal stenosis).

The 15 mm connector is a standardised component to which other airway management devices (e.g. mechanical ventilator, cough assist, nebuliser etc.) can be connected.

Compared to the usage of an endotracheal tube the anatomical dead space is reduced and there is less need for sedation when using a tracheostomy tube. The risk of long-term complications associated with prolonged endotracheal intubation (e.g. vocal cord injuries, formation of granulation tissue in the laryngeal area etc.) can be prevented when using a tracheostomy tube.

Patient Population: The product is intended for adults and adolescents (≥ 12 –21 years).

Clinical Use: The product is intended for mechanically ventilated and self-breathing patients in hospitals, pre-hospitals (EMS), extended care facilities, or outpatient clinics, or home care.

Intended User: The product can be used by medical staff trained in tracheostomy care or individuals trained by professionals.

Indications for Use: The tracheostomy tube is indicated for patients where access to the lower respiratory tract is required by means of a tracheostomy to secure the airway. TRACOE *vario* tracheostomy tubes are single-lumen tubes. Therefore, their application is recommended when there is no acute danger of the tracheostomy tube becoming encrusted or displaced by viscous secretions.

The tracheostomy tube with a high-volume-low-pressure (HVLV) cuff seals the trachea to separate the upper airways from the lower respiratory tract. Therefore, it allows efficient ventilation and reduces influx of subglottic secretions into the lung.

The Minimally Traumatic Insertion System is single use only and is intended for the insertion of the *vario* tracheostomy tube using the Seldinger technique. It can be used for percutaneous dilation tracheostomy or when the tube is changed.

EN

The TRACOE *vario* XL models (REF 451-P, REF 461-P and REF 471-P) are indicated whenever the standard tube length is insufficient, or when it is necessary to position the low-pressure cuff further in the caudal direction (e.g. to stent tracheal stenosis).

The TRACOE *vario* extract tracheostomy tubes with subglottic suction line and cuff (REF 470-P and REF 471-P) are predominantly used for patients producing large amounts of secretions and for whom suctioning of the subglottic space is indicated.

The *vario* extract tracheostomy tube can be used for Above Cuff Vocalisation (ACV).

Single Patient Use and Useful Life: The TRACOE *vario* tracheostomy tube is for single patient use with a useful life of 29 days. The device can be cleaned and re-placed in the same patient during this time period.

The device should not be used for more than 29 days beginning from the initial opening of the sterile barrier. This maximum period of use includes both patient and non-patient (e.g. cleaning) use of the device.

Caution:

A prolonged use of the tracheostomy tube for more than 29 days may result in material safety and biocompatibility issues.

2. General Description

The *vario* tracheostomy tube is made of PVC and provides an artificial airway to the lower respiratory tract.

The product includes a tracheostomy tube, a minimally traumatic insertion system (inserter, guiding catheter with silicone sleeve), an obturator, a fabric neck strap, and lubricating gel which are supplied together within a sterile bag. Adaptors for use with external suctioning devices are only delivered with the subglottic suctioning models (REF 470-P, REF 471-P).

The *vario* tracheostomy tubes are available in different diameters and lengths. The tracheostomy tubes, included with the minimally traumatic insertion system, are cuffed models which are provided with the cuff deflated. The appropriate diameter and length of the tube is determined by the physician.

The tracheostomy tube is radiopaque, due to the incorporated metal spiral reinforcement (REF 450-P, REF 451-P) or from a radiopaque strip available in non-reinforced (clear PVC) models (REF 460-P, REF 461-P, REF 470-P, REF 471-P).

Clinical use of the device in an MR environment is dependent on the product specifications and is described in chapter "MRI Safety Information".

The tracheostomy tube can be used in combination with medical devices that are approved for invasive ventilation through a tracheostoma and are connected via a standard 15 mm connector. The tracheostomy tubes with the subglottic suction line can be used with medical devices approved for subglottic suction.

This product is supplied with an information card, including two detachable labels, which contain

product specific details. These labels will facilitate reordering of the device and its safe use within a MR environment. The labels can be attached to the patient record.

Fold out pages: The image 1 represents the most complex tracheostomy tube model.

1	tube	9	inserter
2	adjustable neck flange	9a	handle with screwing function
2a	lever	10	guiding catheter
2b	push button with spring element	10a	silicone sleeve
2c	wings with eyelets	11	obturator
3	HVLP cuff	12	neck strap
3a	inflation line	13	lubricant gel
3b	pilot balloon with check valve	14	adaptors
3c	female standard Luer connector		
4	scale for axial orientation		
5	standard 15mm connector		
6	suction line		
6a	suction line port with female standard Luer connector		

(1) Tracheostomy Tube:

- All tubes are curved and feature a smooth, round tip at the distal end (inside the patient).
- The tracheostomy tube (1) has a scale for axial orientation (4) behind the 15 mm connector (5) indicating the position of the tube relative to the adjustable neck flange (2). This scale has no measuring function.
- REF 450-P, REF 451-P: The tube has a radiopaque metal spiral-reinforcement.
- REF 460-P, REF 461-P, REF 470-P, REF 471-P: The tube is clear and has a radiopaque stripe.
- TRACOE *vario* extract tracheostomy tube with suction line (REF 470-P, and REF 471-P) allows suctioning of secretions from the subglottic space.
- The standardized 15 mm connector (5) is permanently attached to the tube and is intended for connecting the tracheostomy tube to external devices with a female standardized 15 mm connectors e.g. connection to mechanical ventilation, HME.

(2) Adjustable Neck Flange:

- With the adjustable neck flange (2) the insertion depth of the tracheostomy tube can be adapted to the patient's anatomical requirements.
- The flange includes two flexible wings with eyelets (2c) for attachment of the neck strap (12). The wings can be rotated within a range of approximately 60° to 180°. This permits universal adjustment to meet the patient's requirements.
- The product code (REF), clinical size (size), internal diameter (ID), external diameter (OD), length range (TL) of the tube and MR Safety symbol are all indicated on the neck flange.

(3) High-Volume-Low-Pressure (HVL) Cuff:

- The HVL-cuff (3) is located on the distal end of the tracheostomy tube and directly connected to the inflation line (3a).
- The proximal end of the inflation line includes a pilot balloon (3b), with incorporated self-sealing check valve and a female Luer connector (3c).
- The HVL-cuff is inflated with air only.
- The pilot balloon (3b) displays the cuff resting diameter (CD) and size, where appropriate.

(6–6a) Subglottic Suction Line:

- TRACOE *vario* extract tracheostomy tubes (REF 470-P, REF 471-P) include a subglottic suction line (6) that is integrated into the wall of tracheostomy tube and exits through an aperture at the lowest possible position above the cuff.
- The proximal end of the suction line includes a standard female Luer connector (6a) port for connection to an external accessory device used in subglottic suctioning or for the air/oxygen supply for ACV. For subglottic suctioning, additional adaptors (14) can be used for connection.
- The subglottic suctioning port (6a) can be closed by using the attached cap.

(9–10 a) Minimally Traumatic Insertion System:

- The combination of the inserter (9) and guiding catheter (10) with silicone sleeve (10a) constitute the minimally traumatic insertion system.
- The silicone sleeve bridges the gap between the conical end of the inserter and the distal end of the tube.
- The minimally traumatic insertion system is used for the placement of the *vario* tracheostomy tube using the Seldinger technique.

(11) Obturator:

- The non-perforated obturator (11) has a smooth, round, conical tip at the distal end. The obturator is used for re-insertion of the tracheostomy tube for a tracheostoma that does not require the Seldinger technique.

(12) Neck Strap:

- The neck strap (12) is a soft strip of padded fabric that wraps around the patient's neck.
- The ends of the strap include hook-and-loop fasteners that are inserted through the eyelets of the neck flange to secure the tracheostomy tube in position.
- The frequency of change is determined by the physician or healthcare professional.

(13) Lubricating Gel:

- The lubricating gel (13) can be used for the insertion of the minimally traumatic insertion system or obturator.

Supplementary Products:

- Products, which can be used in combination with the *vario* tracheostomy tube are listed in section "Supplementary Products".

3. MRI Safety Information

MR Unsafe REF 450-P and REF 451-P

Keep the *vario* tracheostomy tube REF 450-P and REF 451-P outside the MRI scanner room.

MR Conditional REF 460-P, REF 470-P, REF 461-P and REF 471-P

Nonclinical testing has demonstrated the TRACOE *vario* tracheostomy tube is "MR Conditional". A patient with this device can be safely scanned in a MR system meeting the following conditions:

- Static magnetic field of 1.5 Tesla (T) or 3.0 T.
- Maximum spatial field gradient of 1900 gauss/cm (19 T/m).
- Maximum MR system reported whole body averaged specific absorption rate (SAR) of 2 W/kg (normal operating mode) and a maximum whole head specific absorption rate (SAR) of 3.2W/kg.
- Quadrature driven transmit body coil only.
- The neck flange must be secured in place with the neck strap.
- The check valve of the tracheostomy tube cuff must be secured to the skin with medical tape, away from the area of MR diagnostic interest.

In non-clinical testing, the image artifact, caused by the check valve, extends (radially) up to 107 mm from the check valve when imaged with a gradient echo pulse sequence and a 1.5 T MR system, and up to 113 mm when imaged with a spin echo pulse sequence in a 3.0 T MR system. Therefore, it is recommended to tape the check valve to the patient's skin away from the area of interest.

Warning:

When used in MR imaging:

- Securely fasten the tube, with a metal-free neck strap, to prevent possible movement while in the MR environment.
- Securely affix the check valve away from the area of interest with standard medical tape to prevent movement within the MR environment.
- MR image quality may be compromised if the area of interest is close to the position of the inflation valve.

4. Contraindications

Trachostomy Tubes:

- Spiral-reinforced models (REF 450-P and REF 451-P) are "MR Unsafe" and must be outside the MR environment.
- The tracheostomy tube cannot be used in conjunction with heat emitting devices, e.g. laser. There is a risk of fire, also toxic gases may form, and the tube may get damaged.
- Not suitable for patients with spasms, which may result in excessive axial forces > 15 N (e.g. neurological spasticity).
- The HVLP cuff should not be inflated, when a speaking valve or an occlusion cap is used and vice versa.
- Neonates, infants, and children (<12 years).

ACV use:

- Patient with new tracheostoma (less than 7–10 days after surgical incision).
- Obstructions in the upper airway that can inhibit the airflow and therefore phonation capabilities.
- Obstructions may lead to pressure rise in the trachea and therefore cause a risk of subcutaneous emphysema.
- Patients with surgical emphysema or infections of the tracheal tissue.
- Patients with unilateral or bilateral paralysis of the vocal cords in median position.

5. General Precautions

- When the product is used together with other medical devices, follow their respective instructions for use. Contact the manufacturer if there are any questions, or if assistance is required.
- Safety precautions must be taken in case of complications during the described procedure.

dures, in order to provide immediate ventilation through alternative airways, (e.g. trans laryngeal intubation, laryngeal mask).

- Optimum oxygen levels must be established in the patient before cannulation or re-cannulation.
- It is strongly recommended that a ready-to-use spare device is kept at the patient's bedside. Store the spare device under clean and dry conditions.
- The product should be inspected for integrity and function prior to use/insertion. Verify that the tube is free of obstruction and the cuff material is not brittle or torn and can be inflated/deflated, absence of kinks, tears or cuts, stable connection between the tube and the neck flange, etc. If the product is damaged, it should be replaced with a new product
- The sterile packaging should be inspected for damage prior to opening. If the packaging is damaged or has been unintentionally opened, the device should not be used.
- While in placement, use or removal of the tracheostomy tube do not use excessive forces.
- Do not use unnecessary force on the tracheostomy tube when connecting to or disconnecting from external devices. This may result in damage of the tracheostomy tube and/or displacement / decannulation.
- Always hold the tracheostomy tube at the base of the 15 mm connector when connecting to or disconnecting from external devices.
- Do not move the adjustable neck flange along the tube axis without unlocking it.
- Lubricating gel might inhibit the locking mechanism of the adjustable neck flange. Ensure that the lubricating gel is applied only on the areas described in chapter "Preparation".
- The cuff pressure can change if nitrous oxide (laughing gas) is used as an anesthetic.
- All parts of the cuff inflation system must be free from strain and kinking during measurement of the cuff pressure, otherwise the manometer may show incorrect pressure values.
- Ensure that all allowed objects (e.g. hand-held manometer) used to inflate the cuff are clean (free of dust, visible particles, and contaminants). Any obstruction of the cuff filling system may result in deflation of the cuff which will reduce efficiency of ventilation or protection from aspiration.
- To avoid damage to the cuff and improve ease of insertion, always ensure that the cuff is deflated prior to insertion with the deflated cuff towards the neck flange.
- Do not move the neck flange with excessive force against the stop near the 15 mm connector. Otherwise the cuff inflation line may be damaged.
- When a manometer and/or a connection tube is attached to the filling line of an inflated cuff, there will always be pressure compensation between cuff and connected device. This will result in a slight pressure loss in the cuff. If necessary, re-adjust the pressure until it is within the optimal range.
- Water inside the Cuff: All HVLP cuffs have a certain degree of permeability to water vapor. Therefore, condensed water vapor may accumulate inside the cuff. If larger quantities of water inadvertently enter the inflation line, it may lead to improper cuff pressure measurement, cuff pressure adjustment, and cuff deflation. In this case, the tracheostomy tube must be replaced.
- During subglottic suctioning, ensure that negative pressure is not excessive and not applied for an extended period in order to avoid drying out of the subglottic area. Intermittent suction is recommended. Closing the cap of the suction line port after suctioning will reduce the drying-out effect. The suction line may be blocked due to accumulated and/or dried secretions inside the suction line or during suctioning of excessive fluid. If the suction line becomes blocked, follow instruction in chapter "Subglottic Suction".
- Improper storage conditions may result in product or sterile barrier damage.

6. Warnings

- Do not use this product if the sterile packaging has been compromised/damaged, e.g. open edges, holes in packaging etc.
- Reprocessing (including re-sterilisation) is not allowed, this may influence the material and function of the product. The products are single use only.
- Modifications of TRACOE products are not allowed. TRACOE will not be responsible for modified products.

- After removing the insertion system, ensure that the silicone sleeve is still located on the guiding catheter. If not, the silicone sleeve (radiopaque) must immediately be removed from the tube or airway.
- During initial placement of a tracheostomy tube immediately stop the ventilation through the upper airways when the cuff of the inserted tracheostomy tube is inflated. This reduces the risk of barotrauma.
- Ensure that the cuff is not punctured by instruments or sharp tracheal cartilage ridges.
- Use only water-soluble lubricating gel for tracheostomy applications, as oil-based gel may damage the tube.
- Ensure that the tube does not become obstructed when applying lubricating gel to the inserter/obturator tip.
- Check the position and function of the tube following insertion. Incorrect placement may result in permanent damage to the tracheal mucosa or minor bleeding.
- Do not move or shift the tube once it is in position, as this may damage the stoma/trachea or lead to insufficient ventilation.
- For correct orientation of the tube and adjustable neck flange, it is essential, that the scale on the tube faces upwards (cranial) and the distal end of the tube is oriented caudal. The information engraved on the neck flange must be readable (TRACOE logo towards patient's chin; see Image 2).
- To avoid damage to the cuff material it should not be in contact with local anesthetics containing aerosols or any ointments, i.e. dexpanthenol.
- Long-term and excessive cuff pressure above 30 cmH₂O (≈ 22 mmHg) poses a risk of permanent damage to the trachea.
- Only fill the cuff with air. Do not fill the cuff with liquids as this would lead to cuff pressure peaks above 30 cmH₂O.
- Insufficient filling (below 20 cmH₂O) of the cuff could result in insufficient ventilation and/or an increased risk of aspiration, which may result in the worst case in VAP (ventilator associated pneumonia) or aspiration pneumonia.
- When repositioning the patient, while in bed, ensure that the patient does not lie on the pilot balloon, as this could increase the cuff pressure and potentially damage the trachea.
- To prevent damage to the stoma or trachea, ensure that the cuff is deflated (empty) prior to insertion or removal of the tube. If it is not possible to deflate the cuff, cut the inflation line with a pair of scissors and remove the air. In this event, the product is defective and must be replaced.
- During air travel alteration of the cuff pressure may occur. Therefore, ensure permanent cuff pressure control.
- Before deflating the cuff ensure that the patient's upper respiratory tract is unobstructed. When applicable, clear the upper respiratory tract of any secretions through suction or patient coughing.
- Make sure that the correct Luer connectors are used for filling the cuff (transparent) and suctioning (white).
- Make sure that the correct Luer connector (white) is used for ACV.
- Ensure that the tracheostomy tube is free of obstructions which may lead to reduction of the provided airflow. Therefore, regular suctioning of the secretion inside the tube is recommended.
- Excessive viscous secretion may lead to dislocation of the tracheostomy tube. Ensure the correct placement of the tube by regularly checking of the tube position and reduce the risk of dislocation by subglottic suctioning of the secretion.
- Use only suction catheters to clear the secretions from the patient's respiratory tract and the tracheostomy tube. Instruments may wedge in the tube and restrict ventilation.
- Regularly check that all connections are secure to prevent an inadvertent disconnection of the tube from external equipment and ensure efficient ventilation.
- Keep the 15 mm connector clean and dry.

- Do not use non-authorized tools to detach 15 mm connector, as this might deform the 15 mm connector.
- Occlusion caps/speaking valves must only be used with a deflated cuff to avoid the risk of suffocation.
- During insertion and removal of the tube a need to cough or bleeding may occur.

7. Side Effects

Typical side effects of tracheostomy tubes use include bleeding, pressure points, pain, stenosis, and skin irritation (e.g. due to moisture), granulation tissue, tracheomalacia, tracheoesophageal fistula, increased secretion, and swallowing difficulties. In case of an adverse event please contact a medical professional immediately.

When using ACV, typical side effects include increased secretion, discomfort, hoarseness, coughing, nausea, or laryngeal drying out due to restoring upper respiratory tract (cleaning / tasting / speaking) functionality.

8. Functional Description

Caution:

- It is strongly recommended that a ready-to-use spare device is kept at the patient's bedside. Store the spare device under clean and dry conditions.
- In case of complications during tube insertion it is also recommended to have a spare device one size smaller available than the tube in use.
- Safety precautions must be taken in case of complications during the described procedures, in order to provide immediate ventilation through alternative airways, (e.g. trans laryngeal intubation, laryngeal mask).

8.1 Preparation

This is a sterile device, which enables use within an aseptic environment.

The size of the tube and appropriate length is determined by a physician.

The following functions must be checked immediately prior to use. If the cuff leaks during use, replace the tube and contact TRACOE Customer Service. If the device fails the initial inspection, repeat the procedure with a new device. Do not discard the device, and follow instructions provided in section "Returns and Complaints".

A. When using the Minimally Traumatic Insertion System

1. Inspect the sterile packaging to ensure it is secure, undamaged and all components are present.
2. Open the package and inspect the device for damages prior to use.
3. Verify that the tube is free of obstruction, the material is not brittle or torn, the cuff is intact, the inflating or suction lines are not kinked, there are no tears or cuts, the connection between the tube and the neck flange is stable.
4. Check the HVLV cuff for leakage by inflating with a hand-held manometer, to a pressure of 50 cmH₂O (≈ 36.78 mmHg). Watch the filled cuff for 1 minute to detect leakage by pressure decrease / cuff deflation. If the cuff is leak tight, remove all the air with a syringe. Shift the deflated cuff material towards the neck flange to facilitate its sliding through the stoma.
5. For adjustment of the tube length, ensure, the orange lever on the right-hand side of the flange is open (lower position "unlocked", see locking symbol on flange). Press the button with spring element and move the flange along the tube. Release the push button when the flange is in position. Push the locking lever upwards ("locked" position) to fix the flange in place (see Image 3).

Caution:

- Do not move the neck flange with excessive force against the stop near the 15 mm connector to prevent the inflation line of the cuff from being damaged.
- For correct orientation of the tube and adjustable neck flange, it is essential, that the

scale on the tube faces upwards (cranial) with the distal end of the tube oriented caudal. The information engraved on the neck flange has to be readable (TRACOE logo towards patient's chin; see Image 2).

6. Check the rotating function of the neck-flange wings and place both wings in a position that will be most comfortable for the patient.
7. Check if the pre-assembled silicone sleeve smoothly bridges the gap between the conical end of the inserter and the distal end of the tube. If a gap is visible, the connection can be tightened by carefully rotating the handle of the inserter.
8. Check if the Seldinger wire (not included), fits smoothly inside the guiding catheter of the minimally traumatic inserter.
9. The silicone sleeve and the end of the tracheostomy tube are then lubricated with the lubricant gel supplied.
10. If appropriate, the neck strap can be attached to the neck flange wings for fixation after insertion of the tube.

B. When using the obturator for re-insertion

Follow steps 1–6 as described above and proceed as follows:

11. Ensure the obturator inside the tracheostomy tube can be easily moved in and out of the tube.
12. Ensure a suction catheter can be easily inserted through the tube.
13. Place the obturator inside the tracheostomy tube.
14. Apply a thin film of lubricant gel to the protruding part of the obturator at the lower part of the tube including the cuff.
15. If appropriate, the neck strap can be attached to the neck flange wings for fixation after insertion of the tube.

8.2 Preparing the Patient

Ensure that the patient is optimally pre-oxygenated immediately before insertion or re-insertion. To facilitate insertion, slightly overextend the patient's neck, if possible.

8.3 Inserting the Tube

A. With the Minimally Traumatic Insertion System

For initial insertion following tracheostomy, the following steps must be carried out subject to bronchoscopic monitoring:

1. The tracheotomised patient has been prepared for cannulation and a Seldinger wire has been inserted into the stoma canal.
2. Insert the proximal end of the Seldinger wire into the distal end of the guiding catheter without pulling the Seldinger Wire out of the trachea.
3. If using the TRACOE *percutan* Seldinger Wire place the proximal end of the guiding catheter at the marking on the Seldinger wire.
4. Insert the tube together with inserter and Seldinger wire through the tracheostoma into the trachea. Take care that the tube, inserter handle, guiding catheter, and Seldinger wire remain aligned and fixed with one hand. Any displacement may result in a gap between the silicone sleeve and tube and resulting in a difficult or aborted insertion.
5. Gently push the tube forward until the neck flange is in contact with the skin surface.
6. Once in place, secure the tube with one hand and withdraw the inserter and guiding catheter together with the Seldinger wire while the tracheostomy tube remains in position within the trachea.

Caution:

- After removing the inserter, ensure that the silicone sleeve is still located on the guiding catheter. If this is not the case, the silicone sleeve (radiopaque) must immediately be removed from the tube or airway.

- If the tube requires additional adjustment unlock the neck flange and adjust accordingly.

B. With the Obturator

The obturator is non-perforated and cannot be used in combination with a Seldinger wire.

1. Prepare the tube and the patient as described in chapter "Preparation" and "Preparation the Patient"
2. When inserting the tube (with the obturator inside) into the tracheostoma, hold the tube at the flange and press the obturator firmly against the 15 mm connector.
3. Gently push the tube forward until the neck flange is in contact with the skin surface.
4. Secure the tube with one hand and remove immediately the obturator after insertion.

Caution:

- If the tube requires additional adjustment unlock the neck flange and adjust accordingly.

8.4 Following Tube Insertion

1. Check if the airway through the tube is unobstructed and if necessary, adjust the position of the tracheostomy tube (e.g. using a bronchoscope) and the neck flange. The correct position of the TRACOE *vario* tracheostomy tube neck flange must be checked regularly.
2. Connect the 15 mm connector of the tracheostomy tube with the respiratory system if ventilation is required.
3. If appropriate: Inflate the cuff of the tracheostomy tube with air through the Luer connector located at the pilot balloon.
4. If necessary, the wings of the neck flange can be re-adjusted.
5. To prevent the tube dislocation, secure the tube in place with the neck strap.
6. It is recommended that a dressing is placed between the tracheostoma and the adjustable flange to prevent irritation of the skin underneath the flange.
7. Re-check the cuff pressure to make sure that the cuff has not been damaged during the insertion.

8.5 Inflating the Cuff

Option 1: In place of a standard syringe for inflating the cuff, we recommend the use of a hand-held manometer. Adjust the cuff pressure to the individual ventilation therapy and check it at regular intervals. Typically, the pressure should be between 20 cmH₂O (≈ 15 mmHg) and 30 cmH₂O (≈ 22 mmHg).

Option 2: Use a TRACOE smart Cuff Manager to maintain the cuff pressure within the range of 20 to 30 cmH₂O through passive control. Attach the male Luer of the TRACOE smart Cuff Manager to the female Luer of the check valve of the tracheostomy tube. Inflate the TRACOE smart Cuff Manager using a standard syringe according to the respective IFU.

Caution:

- When repositioning the patient, while in bed, ensure that the patient does not lie on the pilot balloon, as this could increase the cuff pressure and potentially damage the trachea.

8.6 Connecting/Disconnecting External Equipment

To connect to external equipment or accessories (e.g. ventilator) firmly hold the base of the 15 mm connector and gently push the connection end of the external device until it is securely attached to the tracheostomy tube. If in doubt, twist the connection end on and off several times, in order to confirm the amount of force needed to ensure the connection is secure and the external device can be easily disconnected at a later time.

If disconnection is difficult, use a standardized disconnect wedge (not supplied) to uncouple the tracheostomy tube from external equipment or accessories (see Image 4) by sliding the opening

of the disconnect wedge between the 15 mm connector and external device until the two devices are separated, chapter “Supplementary Products”.

Caution:

- Do not use unnecessary force on the tracheostomy tube when connecting to or disconnecting from external devices. This may result in damage of the tracheostomy tube and/or displacement / decannulation.

8.7 Subglottic Suction

1. To perform intermittent suctioning, remove the cap of the subglottic suction line Luer connector.
- 2a. Manual suctioning can be carried out using a syringe.
- 2b. An active suction device can be connected using the adaptors, (see image 5).
3. Following subglottic suctioning, reseal the suction line Luer connector with the cap.

Caution:

- If the suction channel is obstructed, it can be cleared by inflation of air/ oxygen (recommended 3–6 l/min; max. 12l/min) or it can be rinsed with saline solution (recommended 2–3 ml). Do not exceed the recommended limits and take care of the patient’s individual tolerability. The following side-effects could occur: Accumulation of potentially contaminated secretions, discomfort, nausea and retching, excessive secretions.
 - Before rinsing the suction line, make sure that the cuff is sufficiently inflated.
 - Remove the applied saline immediately after rinsing the suction line.
 - If the suction line doesn’t get cleared, the tube must be changed.

8.8 Above Cuff Vocalisation

Caution:

- ACV must be performed by professional personnel.

ACV is used to provide phonation capabilities for the patient. Therefore, it must be adjusted to the individual patient’s needs and abilities. It is essential that the patient is instructed and involved in every step of ACV to ensure cooperation and good results during the application.

Before using ACV ensure that the patient is wearing a tracheostomy tube with permanently inflated cuff and does not tolerate cuff deflation. If needed, air can be humidified before inflation through the subglottic suction line to prevent the laryngeal mucosa from drying out.

1. Explain the planned procedure to the patient. Indicate possible adverse reactions and clarify patient’s questions.
2. Verify that the upper airways are not obstructed.
3. Clear the subglottic space from secretions using subglottic suctioning.
4. Verify that the suction line is not obstructed.
5. Connect properly the adjustable air or oxygen supply via a fingertip connector to the female Luer connector of the subglottic suction line. Alternatively, other devices for interruption of the permanent airflow may be used (e.g. Y-connector).
6. Inflate air slowly into the upper airways of the patient starting with 1 l/min and slowly rising to a typical flow rate of 3–6 l/min depending on the patients’ requirements. To may reduce laryngeal mucosa from drying out, flow rates must not exceed 12 l/min. Use the fingertip connector to limit the air flow time. This timeframe should be adapted to the patient’s exhaling rhythm. Adjust airflow and time within the comfort zone of the patient.
7. Monitor the patient’s reaction and adjust parameters (flow and time of airflow) as necessary.
8. When the session is finished, turn off the air flow and disconnect the equipment from the subglottic suction line connector and replace the cap.

Caution:

- The airflow through the upper airways may irritate the patient or may lead to increased secretions, coughing, nausea or retching.
- If the voice sounds gruff, repeat subglottic suction to clear the airway.
- Adjust the duration of a single ACV session to the capabilities/endurance of the patient.
- Use short sessions of ACV to prevent drying of the laryngeal mucosa.
- Regularly monitor patients with tracheostoma by medically trained staff.

8.9 Deflating the Cuff

Before deflating the cuff, ensure that as little secretions as possible enter the lower respiratory tract, e.g. by subglottic suctioning, suctioning through the tube. To deflate the cuff, attach a syringe (with the plunger pushed in) to the female Luer connector of the pilot check valve. Pull the plunger back until all air is removed from the cuff. The cuff must be deflated (empty) prior to removal of the tracheostomy tube.

Caution:

- When removing the air from the cuff, pay attention to the volume of the air removed. This services as a reference for the integrity of the system for further cuff inflation.

8.10 Removing the Tube

In case of the tube change, prepare the replacement tube as described in chapter "Preparation".

1. Slightly overextend the patient's neck, if possible.
2. Deflate the cuff (see chapter "Deflating the Cuff").
3. Secure the neck flange, while loosening the neck strap.
4. Firmly hold the base of the 15 mm connector and gently pull the tracheostomy tube from the stoma.

If necessary, suctioning of secretions through the tube may be helpful to prevent infiltration into the lower respiratory tract.

5. Following removal, the tube should be cleaned as soon as possible to prevent encrustation of fluids.
6. If the product is damaged, do not reuse the tube. Please inform TRACOE Customer Service with the REF and LOT number, do not discard the tube and follow instruction in chapter "Returns and Complaints".

9. Care and Cleaning**Caution:**

- The device should not be used for more than 29 days beginning from the initial opening of the sterile barrier.
- This maximum period of use includes both patient and non-patient (e. g. cleaning) use of the device.
- The product should be inspected for integrity and function prior to re-use/re-insertion.

Cleaning of the tracheostomy tube and obturator is intended to remove any bodily fluids or encrustation that may inhibit its clinical use.

Please take care to hold the tube after cleaning at the 15 mm connector and the obturator at its handle.

The following instruction for manual cleaning applies to all TRACOE *vario* models and sizes:

1. To clean the tube and obturator, rinse the devices separately under body warm (max. 37 °C / 98.6 °F) potable water until they are visibly clean and free of encrustations.

2. Particular attention should be taken to ensure the inside of the tube, and as appropriate the subglottic suction line (REF 470-P, REF 471-P), are thoroughly rinsed.
3. For removal of residual debris TRACOE approved brushes or swabs can be used, see “Supplementary Products”.
4. Alternatively, the TRACOE tube clean cleaning kits (see “Supplementary Products”) can be used in accordance to their respective instructions for use.
5. After cleaning, rinse the tube with potable or distilled water.
6. If the tube is not visually clean after rinsing then:
 - repeat rinsing until it is visibly clean, or
 - repeat the cleaning using the TRACOE cleaning kits, or
 - safely dispose of the tracheostomy tube.
7. All areas of the tube and obturator should be inspected, in adequate light, to ensure the device is free of contaminants and encrustations.
8. Following the cleaning process, place the tube and obturator on a clean lint-free dry towel and air dry in an area free of airborne contaminants.
9. The tube and obturator are considered dry when there is no visual evidence of residual water. Please check, that the inner of the cuff is dry.
10. Finally, a visual and functional inspection prior to re-insertion should be performed to verify that the tube and obturator are not damaged (also see chapter “Preparation”).

Caution:

- The tracheostomy tube and obturator should be cleaned immediately after removal from the stoma to prevent drying of soil and contaminants.
- When cleaning, take care not to damage the cuff or the inflating line.
- The frequency of cleaning must be defined by the physician but must not exceed the allowed frequency.
- Maximum allowed cleaning frequency is once a day, otherwise biocompatibility and material stability could be impaired.
- The tubes must never be cleaned using agents or procedures which are not specified in this instruction.
- The tracheostomy tube is single patient use, therefore, it must be returned to the same patient.
- Failure to clean the device properly can result in damage to the tube, an increase in air resistance due to obstructions, or irritation/inflammation of the tracheal stoma.
- Since the upper respiratory tract is never free from microorganisms, even in healthy individuals, we do not recommend the use of disinfectants.
- The minimally traumatic inserter and guiding catheter are single-use and not allowed to be cleaned and reused.

10. Storage

- a) Store the TRACOE products in their original packaging according to the conditions displayed on the packaging.
- b) Store cleaned tracheostomy tubes in a clean covered container, within a clean and dry location and away from sunlight. Re-insert the tracheostomy tube as soon as possible. Improper storage conditions may result in tube damage or contamination. Do not store the cleaned devices for more than 29 days from first use.

11. Packaging

The product is provided sterile (with ethylene oxide) which allows application under sterile conditions. TRACOE tracheostomy tubes do not require a sterile environment during normal use or cleaning.

12. Disposal

Used products are to be disposed of in accordance with the national regulations, waste management plans, or clinical procedures governing biohazardous waste materials, e.g. the direct disposal in a tear and moisture resistant secure bag or container, which is routed to the local waste disposal system for contaminated medical products.

For further recommendations, contact your hygiene officer in health facilities, or the local waste management for homecare use.

13. Returns and Complaints

Returned products, that have been used, will only be accepted if TRACOE has agreed to the return and a completed decontamination certificate with complaint report is enclosed with the device. These forms are available either directly from TRACOE medical, or via the website www.tracoe.com.

If the device is involved in a reportable incident, as defined in local medical device legislation, please contact TRACOE medical (complaints@tracoe.com), and the appropriate regulatory body in the country of use.

14. Supplementary Products

TRACOE *vario* Tracheostomy Tubes with Minimally Traumatic Insertion System are also available as TRACOE *experc* Set (REF 420 – REF 425) in combination with TRACOE *experc* Dilation Set (REF 520).

The TRACOE *percutan* Seldinger Guide Wire is part of the above-mentioned sets. It is also available separately with guiding catheter (REF 517) or without guiding catheter (REF 518).

14.1 Recommended Products

- Syringes with standard male Luer connector
- Cuff pressure monitors for HVLP cuffs with standard male Luer connector (e.g. TRACOE cuff pressure monitor REF 720)
- TRACOE smart Cuff Manager (REF 730 and REF 730-5)
- TRACOE Lubricating Gel (REF 677) and other sterile water-soluble lubricating gels for tracheostomy applications
- Neck straps (e.g. REF 903-F, REF 903-E, and REF 903-D)
- Disconnecting wedges for tracheostomy / endotracheal tubes with 15 mm connectors
- TRACOE *technic* connection tubes including Luer connector with integrated check valve
- Humid Moist Exchangers (HME) with a standard male 15 mm connector

14.2 Optional Products

- Speaking valves and occlusion caps with a standard male 15 mm connector
- TRACOE tube clean cleaning kit (REF 930-A / -B), including TRACOE tube clean cleaning powder (REF 932)
- TRACOE tube clean cleaning swabs (REF 935, and REF 936), and TRACOE tube clean cleaning brushes (REF 938-A/-B/-C, and REF 940-A/-B/-C/-D)
- TRACOE *care* Shower Guard (REF 915)
- Dressings and compresses
- TRACOE *care* Protective Bibs (REF 919-A, -B, -C)
- TRACOE *care* Protective Scarves (REF 921-A to -E)
- TRACOE *care* Protective Roll-Necks (REF 923-A to -G)

15. General Terms and Conditions

The sale, delivery and return of all TRACOE products shall be affected exclusively on the basis of the valid General Terms and Conditions (GTC), which are available either from TRACOE medical GmbH or on our website at www.tracoe.com.

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